



RF Listening Device

How-To Guide

Step-by-step setup for the 72 MHz assistive listening receiver



Using Headphones — Standard Wired Setup

1 Power on the receiver

Locate the **Power button** on the top edge of the device. Press and hold it until the device turns on.



Device overview — Power & Volume controls

2 Plug in your headphones

Insert the headphone plug into the **3.5 mm audio jack** on the top left or right corner of the device. Push until it clicks into place.



Plug the headphone cable into the jack on the top of the device

3 Put on the headphones

Place the headphones over your ears. The **L** (left) and **R** (right) labels are marked on each ear cup.

4 Adjust the volume

Use the buttons labeled **▲ Volume Up** (upper half) and **▼ Volume Down** (lower half) to reach a comfortable listening level.

Tip: Start at a low volume and increase gradually to protect your hearing.

5 Power off when finished

Press and hold the **Power button** until the device turns off. Unplug and return all equipment to staff.

Using a T-Coil / Neck Loop

For hearing aids with telecoil

1 Power on the receiver

Press and hold the **Power button** on the top edge of the BWH ALS device until it turns on.



Device overview — Power & Volume controls

2 Connect the neck loop

Take the neck loop accessory. Plug **both ends** of the loop cable into the two ports on the sides of the receiver — one plug per side.

 **Note:** Both connectors must be fully inserted for the loop to work correctly.

Loop plugged into both sides; headphones optional



Make sure that t-coil device is within the loop

3 Place the loop around your neck

Drape the loop around your neck so it rests on your chest. The receiver hangs at the center. **Keep your hearing aid's T-Coil (telecoil) setting active** and ensure the hearing aid is within the loop area.

Tip: Switch your hearing aid or cochlear implant processor to its T or Telecoil program before use.

4 Optionally add headphones

If you also wish to use headphones simultaneously, plug them into the **3.5 mm jack** on the top of the device (the same port labeled "Plug in headphones here").

5 Adjust volume

Use the ▲ / ▼ volume rocker on the front of the device. Audio is transmitted wirelessly via 72 MHz RF to the receiver.

6 Power off when finished

Press and hold the **Power button** until the device turns off. Disconnect the neck loop and return all equipment to staff.

Device Quick Reference

- **Volume rocker:** ▲ upper half = up, ▼ lower half = down
- **Headphone jack:** top left or right corner of the device
- **Loop ports:** one on each side of the device
- If there is no audio, ensure you have pressed the power button and volume is turned up
- Contact a staff member if you experience any difficulty

BWH ALS Assistive Listening Program · For assistance, please speak with a staff member